P256

P264A 
P265A 

P260

Invacare®Silencio (P256), Invacare®Prompto (P264A, P265A), Invacare®Foldeo (P260)

en	Cane	
	User Manual	2
da	Stok	
	Brugsanvisning	2
es	Bastón	
	Manual del usuario	3
fr	Canne	
	Manuel d'utilisation	4
hr	Štap	
	Upute za uporabu	5
it	Bastone	
	Manuale d'uso	6
lt	Lazda	
	Naudojimosi instrukcija	7
nl	Wandelstok	
	Gebruiksaanwijzing	8
pt	Bengala	
	Manual de utilização	9
de	Gehstock	
	Gebrauchsanweisung	10

Figures

1

2

3

	I	H
		
P256	810 -1010 mm	100 kg
P260	850 - 950 mm	100 kg
P264A	810 -1010 mm	100 kg
P265A	810 -1010 mm	100 kg

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

General

Intended use

The cane is intended to be used solely as a walking aid.

The cane is not designed to take the total weight of the user. For maximum user weight see table on front page.

The cane can be used indoors and outdoors.

Safety

WARNING! **Risk of injury**

- Always ensure that the product is suitable for your requirements. In case of doubt, consult a healthcare professional.
- Before using the cane, check that it is in proper condition and has no defects.
- Check the rubber tips regularly for rips, tears, cracks or wear. In case of damage, replace the rubber tips immediately.
- Take particular care when walking on slippery or wet ground.
- Take particular care, if the cane is exposed to temperatures above 37°C or below 0°C or to high humidity before use.
- Do not store the cane outdoors.

Labels and symbols on the product

This section refers to the serial label in figure 1 on the front page.

Ⓐ Product name
Ⓑ Serial number
Ⓒ For indoor and outdoor use
Ⓓ Refer to user manual
Ⓔ Date of manufacture
Ⓕ Manufacturer
Ⓖ Part number
Ⓗ Max. user weight
Ⓘ Height adjustment
Ⓢ This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Warranty information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

Limitation of liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorized modifications and/or use of unsuitable spare parts

Usage

Adjusting the height

This instruction refers to figure 2 on the front page.

1. Press the snap button.
2. Slide the extension up or down until the desired height is achieved.
3. Allow the snap button to click into the next hole in the cane.
Make sure the snap button is fully engaged in the hole. There will be an audible "click".

Folding and unfolding (P260)

This instruction refers to figure 3 on the front page.

Folding:

1. Pull the bottom tube out of the next tube and fold it aside. Repeat for the remaining tubes.
2. Use the transport strap to keep the tubes folded.

Unfolding:

1. Hold the folded tubes in one hand and the handle with the other hand. Release one tube after the other and let them join.
2. Make sure that all tubes are joined properly.

Maintenance and care

Maintenance should be done at least every six months.

- DO NOT use an abrasive detergent, cleaner or cloth to clean or dry the product.
- We recommend regular cleaning of the product. Wipe all surfaces with a damp cloth and allow all surfaces to dry before use.
- We recommend regular inspection of parts for deformation, corrosion, breakage wear or compression. Replace the product, if any of these conditions exist.
- If the hand grip is loose, DO NOT use the product. Contact your Invacare dealer.
- If rubber tips are worn or cracked, replace the rubber tips immediately.

After use

Disposal

Waste disposal/recycle must comply with the laws and regulations for waste handling in each country.

Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the actions, listed in the Maintenance chapter.

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© Invacare Corporation. All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare Rea AB